

## B. Graczyk, Inc. Universal™ / Surgical Eye Care Products

Mixing Universal Concentrated Surgical Instrument Cleaner and Lubricant for Use:



- Mark Container of Deionized / Distilled Water as Universal Cleaner.
- Loosen the Cap on the smaller chamber of Universal Cleaner Bottle.
- (40-810 960 ml Calibrated Bottle Illustrated.)
- Squeeze the center of the bottle gently and fill the measuring chamber to the desired level.

**Recommended Dilution Ratio:**

**Deionized / Distilled Water.**  
 3.8 Liter / 1 US Gallon  
 1.9 Liter / ½ US Gallon  
 0.95 Liter / 1 US Quart

**Recommended Dilution Ratio:**

**Universal Cleaner Milliliter (ml)**  
 30 ml Cleaner  
 15 ml Cleaner  
 7.5 ml Cleaner

- Pour measured Universal Cleaner into the Container of Deionized / Distilled Water.
- Replace Cap on Deionized / Distilled Water Container.
- Several quick shakes will completely dilute the Universal Cleaner.
- The Universal Cleaner has a stable shelf life for 5 plus years.